



ROHO® Contour Base

Product Information



Sizing Guide*

Contour Base Item Number	Contour Base Width x Depth	Standard Cushions** 1R / 2R / ENH***	Select Cushions** QS / CS***	MOSAIC Cushions****
BASE1313	13.25 in. x 13 in. 33.5 cm x 33 cm	77, 78, 87	77, 78, 87	N/A
BASE1515	15.25 in. x 15 in. 38.5 cm x 38 cm	88, 89, 98	88, 89, 98	MOSAIC1616C, MOSAIC1616HD
BASE1715	17 in. x 15.25 in. 43 cm x 38.5 cm	98, 99, 108, 109	98, 99, 108, 109	MOSAIC1816C, MOSAIC1816HD
BASE1717	17 in. x 17.5 in. 43 cm x 44.5 cm	910, 1010	99, 910, 109, 1010	MOSAIC1818C, MOSAIC1818HD
Contour Base height: 0.38 in. (1.0 cm)				

* Cushion cells should not extend past the perimeter of the Contour Base.

** Suggested base sizes for use with neoprene cushions; dimensions coordinate with cushion item numbers, which are based on the number of cells. (Example: 1R1011C = 10-cell width x 11-cell depth)

*** Item number prefix

**** Product may not be available in all countries. Please check with your ROHO distributor.

Intended Use:

The ROHO® Contour Base (Contour Base) is a contoured, molded base that is intended to provide a solid foundation for use with most ROHO cushions in order to prevent the hammocking effect of wheelchair sling seats. There is no weight limit, yet the Contour Base must be properly sized to the individual. ROHO, Inc. recommends that a clinician experienced in seating and positioning, be consulted to determine whether the Contour Base is appropriate for the individual's particular seating needs.


Placement:

Place the Contour Base inside the cover, between the bottom of the cushion and the bottom of the cover.

Cleaning and Disinfecting:

To Clean: Hand wash with mild detergent in warm water, using a sponge, soft cloth, or scrub brush. Rinse thoroughly. Dry thoroughly with a clean, soft cloth, or air dry.

To Disinfect: Hand wash with a general disinfectant, or 1 part household liquid bleach per 9 parts warm water, using a sponge or soft cloth. Rinse thoroughly. Dry thoroughly with a clean, soft cloth, or air dry.

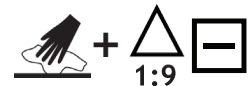

Manufactured by ROHO, Inc., 100 N. Florida Ave., Belleville, IL 62221-5429 U.S.A. • cc@roho.com
U.S. 800-851-3449 • Fax: 888-551-3449 Outside U.S. 618-277-9150 • Fax: 618-277-6518

12-Month Limited Warranty: For additional warranty information, contact Customer Care or your country's ROHO distributor.

Products in this document may be covered by one or more U.S. and foreign patents and trademarks. For further details please refer to www.roho.com.

ROHO® is a registered trademark of ROHO, Inc.

ROHO, Inc. has a policy of continual product improvement and reserves the right to amend this document. A current version of this document is available at www.roho.com.

© 2013, 2014 ROHO, Inc.



T20227 Rev. 6/14



The Quality Management System of ROHO, Inc. is certified to ISO 9001:2008 and ISO 13485:2003.