

# ROHO® AGILITY™ Headrest Adapter Plate **AG-HR-BRKT**



This document is provided as a supplement to the ROHO AGILITY Back Systems Operation Manual and does NOT replace any information in that manual unless otherwise noted.

SUPPLIER: This supplement must be given to the user of this product.

USER: Before using this product, read instructions and save for future reference.

## DESCRIPTION OF USE

The ROHO AGILITY Headrest Adapter Plate (Headrest Adapter) is an optional accessory intended to be used with ROHO AGILITY Back Systems (AGILITY) to attach commercially-available wheelchair headrests.

ROHO, Inc. recommends that a clinician, such as a doctor or therapist experienced in seating and positioning, be consulted to determine if the Headrest Adapter is appropriate for the individual's particular seating needs.

## INFORMATION FOR SAFETY

### ! WARNINGS

- **DO NOT** install a Headrest Adapter on an AGILITY without first reading and understanding this supplement. If assistance is needed, contact a clinician, ROHO provider or ROHO, Inc.
- **DO NOT** attach, remove or adjust the Headrest Adapter when an individual is seated in the chair.
- **Check** the Headrest Adapter for wear, breakage and tightness daily. Retighten loose bolts following the instructions included. **DO NOT** continue to use the Headrest Adapter if any of the components cannot be tightened or are broken.

### MOTOR VEHICLE TRANSPORTATION




**CAUTION:** Whenever possible, the individual in the wheelchair with an AGILITY should transfer into a manufacturer-installed vehicle seat and use the vehicle's crash-tested occupant restraint system.

AGILITY with Quick Release Attachment Hardware and Headrest Adapter has been dynamically tested for use in a motor vehicle by a third-party testing facility. Testing was conducted in accordance with standard RESNA WC-4:2012, Section 20: *Wheelchair Seating Systems for Use in Motor Vehicles* and concluded that AGILITY with the Headrest Adapter meets all applicable performance criteria for wheelchair back supports in 5.1 of WC20.




**WARNING:** AGILITY with a Headrest Adapter must be correctly and securely installed on the wheelchair following all instructions and warnings provided in this supplement and in the AGILITY Back Systems Operation Manual.


## PARTS DETAIL - PACKAGE CONTENTS

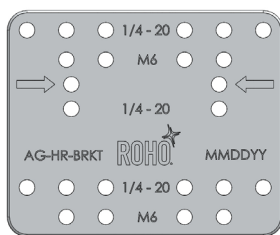

Headrest Adapter

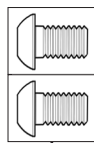

10mm M6-1.0 Bolts

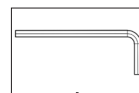

4mm Hex Key



Operation Manual Supplement

ROHO, Inc. has a policy of continual product improvement and reserves the right to amend this document. The current version of this document is available at [www.therohogroup.com](http://www.therohogroup.com).

© 2013 ROHO, Inc.


The following are trademarks and registered trademarks of ROHO, Inc.: ROHO® and AGILITY™.

Products in this document may be covered by one or more U.S. and foreign patents and trademarks. For further details please refer to [www.therohogroup.com](http://www.therohogroup.com).



## PRODUCT SPECIFICATIONS\*

<b>Material:</b>	Black Anodized Aluminum	
<b>Weight:</b>	0.25 lbs. (0.13 kg)	
<b>Width x Length x Depth:</b>	3.5 in. x 2.9 in. x 0.3 in. (89mm x 73mm x 8mm)	
<b>Headrest Compatibility:</b>	Compatible with OEM headrests that have the following center to center bolt spacing:	
	<b>1/4-20 Thread</b>	<b>M6 (1.0) Thread</b>
<b>Horizontal:</b>	1 in., 1-7/8 in., 2 in. and 3 in.	25.4mm and 50.8mm
<b>Vertical:</b>	2 in.	50.8mm
<b>*NOTE:</b> All measurements are approximate.		

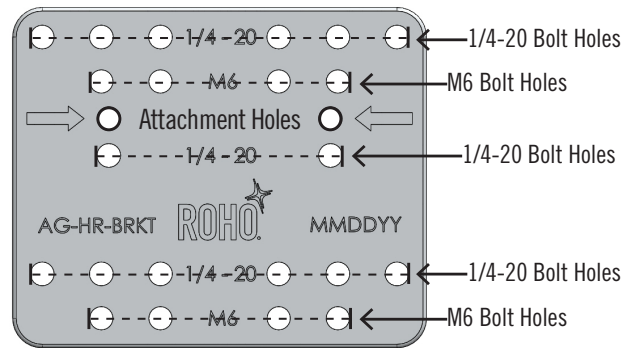

## HEADREST ADAPTER ATTACHMENT


1




Remove the AGILITY cover to allow access to the pre-drilled holes. After Headrest Adapter attachment is complete, the AGILITY cover may be re-installed.

Refer to AGILITY Back Systems Operation Manual - COVER REMOVAL AND INSTALLATION and ACCESSORIES.


2

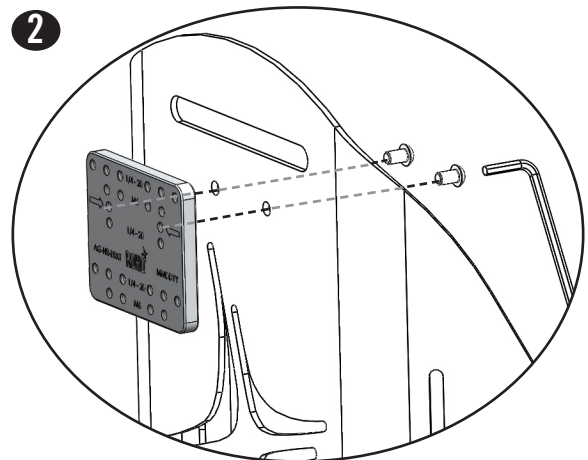

3

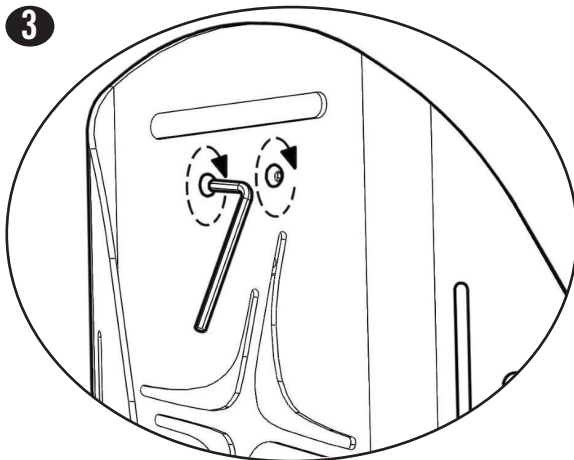

4

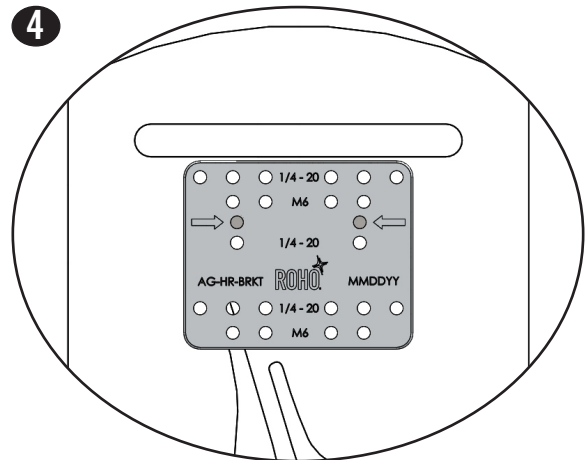

For detailed information regarding CLEANING AND DISINFECTING and LIMITED WARRANTY of back shell hardware, refer to the AGILITY Back Systems Operation Manual.



Manufactured by ROHO, Inc.  
**Administrative Offices:** 100 North Florida Avenue, Belleville, Illinois 62221-5429  
**Mailing/Shipping Address:** 1501 South 74th Street, Belleville, IL 62223-5900, USA  
[www.therohogroup.com](http://www.therohogroup.com)  
 U.S.A.: 800-851-3449, Fax 888-551-3449 e-mail: [cc@therohogroup.com](mailto:cc@therohogroup.com)  
 Outside the U.S.A.: 618-277-9150, Fax 618-277-6518 e-mail: [intl@therohogroup.com](mailto:intl@therohogroup.com)

Item # T20222  
 Revision: 07/16/13