## **RETROBACK™** **Wheelchair Back Support**

### **Operations Manual**

**SUPPLIER:** THIS MANUAL MUST BE GIVEN TO THE USER OF THIS WHEELCHAIR BACK SUPPORT.

**USER:** BEFORE USING THIS WHEELCHAIR BACK SUPPORT, READ THESE INSTRUCTIONS AND SAVE FOR FUTURE REFERENCE.

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## **TABLE OF CONTENTS**

|   |             |
|---|-------------|
| <b>Intended Use</b>                             | <b>1</b>    |
| <b>Precautions</b>                              | <b>1</b>    |
| <b>Parts Detail</b>                             | <b>2</b>    |
| <b>Sizing Table</b>                             | <b>2</b>    |
| <b>Attachment Instructions</b>                  | <b>3</b>    |
| <b>Cover Removal &amp; Replacement</b>          | <b>4</b>    |
| <b>Cleaning and Disinfecting</b>                | <b>5</b>    |
| <b>Return, Repair &amp; Contact Information</b> | <b>5</b>    |
| <b>Warranty</b>                                 | <b>6</b>    |
| <b>Spanish</b>                                  | <b>7-12</b> |

ROHO, Inc. has a policy of continual product improvement and reserves the right to amend specifications presented in this manual.

The Retroback™ Wheelchair Back Support is manufactured in the U.S.A. by ROHO, Inc.  
100 N. Florida Avenue, Belleville, IL 62221-5429  
U.S.A.: 800-851-3449, Fax 888-551-3449  
Outside the U.S.A.: 618-277-9150, Fax 618-277-6518  
www.therohogroup.com • cs@therohogroup.com

© 2008 ROHO, Inc.

The following are trademarks and registered trademarks of ROHO, Inc.: ROHO®, DRY FLOATATION®, Retroback™ and shape fitting technology®.  
Velcro® is a registered trademark of Velcro Industries B.V.

## INTENDED USE

The Retroback Wheelchair Back Support (“Retroback”) is a wheelchair back product designed to fit over the wheelchair sling back upholstery on a range of wheelchair sizes. (See the Sizing Table included in this manual.) The Retroback is composed of a foam pad, a cover, and a steel clip that hooks to a wheelchair sling back. The Retroback can be used with folding or fixed frame wheelchairs.



The Retroback is not a replacement for the original manufacturer’s wheelchair sling back. It is intended to be an add-on product to sling-back wheelchairs that rests between the user and the sling back to promote a more upright posture resulting in improved positioning of the pelvis and trunk. The Retroback is intended to improve the user’s comfort and basic positioning needs. It is designed to be easily attached and removed from a wheelchair without tools or hardware. It is not intended to provide lumbar or lateral support and is not intended to be used with a solid back.

## PRECAUTIONS

- Before using the Retroback, ROHO, Inc. recommends a clinician evaluate the need for anti-tipping devices. As with any addition to your wheelchair, the Retroback may affect the stability of the wheelchair.
- **DO NOT** use the Retroback as a replacement for the original manufacturer’s wheelchair sling back.
- Remove the Retroback from the wheelchair prior to folding the wheelchair.
- The Retroback must be assembled correctly and attached to the wheelchair as described in this manual.
- The Retroback should be sized correctly to the wheelchair and will not fit every wheelchair. The Retroback decreases the seat depth. Refer to the Sizing Table in this manual and consult with a clinician or supplier providing the equipment.
- **DO NOT** use the Retroback to lift the wheelchair.
- The Retroback maximum user weight limit is 300 lbs. (136 kg).
- The Retroback has not been tested for use in a motor vehicle. Always have the user transfer to an approved vehicle seat.
- Protect the Retroback from blunt force shocks that may cause damage and/or breakage.
- Altering the Retroback will void the warranty.
- **DO NOT** hang personal items from the back clip handle.

## PARTS DETAIL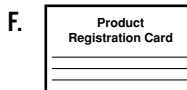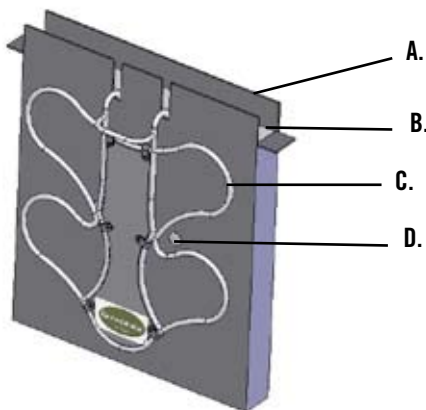

**A. Cover:** The cover consists of a front foam section and a back clip section. The top has a three section Velcro® enclosure. The front material is 100% water resistant nylon and the sides are 100% polyester knit spacer fabric. The back and the section divider are 100% polyurethane coated nylon knitted fabric.

**B. Foam Pad:** Provides additional support to the user's pelvis & trunk. Made of polyurethane foam.

**C. Back Clip:** Chrome plated steel clip. A handle near the top aids in removal of the Retroback from the wheelchair.

**D. Snaps:** Holds back clip in place.

**E. Operations Manual:** Instructions on Retroback attachment, care and other important information.

**F. Product Registration Card:** Complete product registration card and mail to ROHO, Inc. or register online at [www.therohogroup.com](http://www.therohogroup.com).


### Sizing Table

| Item Numbers | Retroback Cover & Foam Pad Size (Width x Height x Depth)  | Fits Wheelchair Widths* |                     | Fits Wheelchair Heights |                     |
|--------------|---|-------------------------|---------------------|-------------------------|---------------------|
|              |   | min.                    | max.                | min.                    | max.                |
| RETRO16C     | 13 in. x 14 in. x 1.5 in.<br>(33.0 cm x 35.5 cm x 4.0 cm) | 15 in.<br>(38.0 cm)     | 16 in.<br>(40.5 cm) | 16 in.<br>(40.5 cm)     | 18 in.<br>(45.5 cm) |
| RETRO18C     | 15 in. x 14 in. x 1.5 in.<br>(38.0 cm x 35.5 cm x 4.0 cm) | 17 in.<br>(43.0 cm)     | 18 in.<br>(45.5 cm) | 16 in.<br>(40.5 cm)     | 18 in.<br>(45.5 cm) |
| RETRO20C     | 17 in. x 14 in. x 1.5 in.<br>(43.0 cm x 35.5 cm x 4.0 cm) | 19 in.<br>(48.5 cm)     | 20 in.<br>(51.0 cm) | 16 in.<br>(40.5 cm)     | 18 in.<br>(45.5 cm) |

\* All measurements taken are from outside of back post to outside of back post.

The average weight of an assembled Retroback is 3 lbs. (1.4 kg).


## **ATTACHMENT INSTRUCTIONS**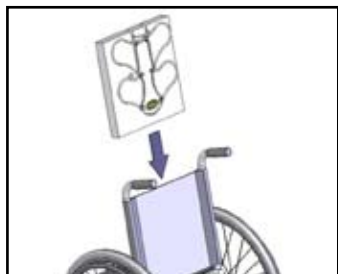

### **To Attach the Retroback to a Wheelchair:**

1. Orient the Retroback with the foam pad side facing towards the front of the wheelchair, the back clip facing towards the back seat sling, and the Velcro side of the cover facing up.
2. Slide the back clip down over the wheelchair sling back so the sling is between the nylon back side of the cover and the back clip.
3. Push the Retroback down until the top of the back clip rests on the center of the top edge of the wheelchair sling back.


The Retroback is now ready for use.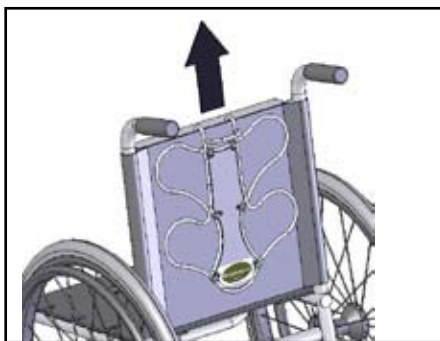

### **To Remove the Retroback from a Wheelchair:**


1. Grasp the handle at the top of the back clip and pull the Retroback straight up until it is clear of the wheelchair sling back.

## **COVER REMOVAL & REPLACEMENT**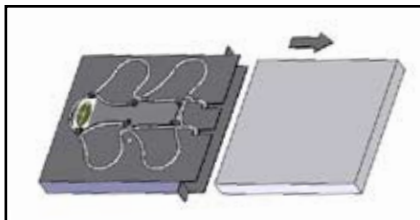

### **To Remove The Cover:**

1. Remove the Retroback from the wheelchair.
2. Pull open the Velcro flap sections on the top edge of the Retroback cover.
3. Pull the foam pad out of the cover's front foam section.
4. Unsnap both of the snaps holding the back clip in place.
5. Slide the back clip out of the cover's back clip section.

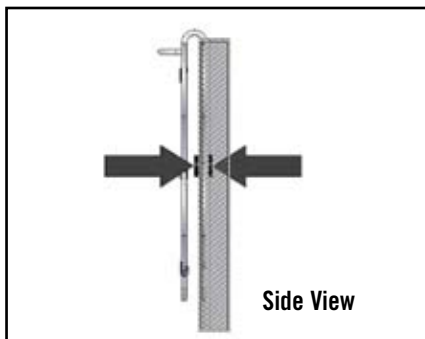### **To Replace The Cover:**

1. Lay the cover on a flat surface with the Velcro top open and the back clip section facing up (two metal snaps should be visible).
  2. Hold the back clip so the Retroback logo is facing up. Slide the open bottom of the back clip into the cover's back clip section so the non-logo side of the back clip rests between the snaps. Note: The side of the clip with the Retroback logo should still be visible on the outside of the cover. Press the snaps together to entrap the back clip (see diagram for snap location).
  3. Turn the cover over so the clip is now facing down. Slide the foam pad into the front foam section. Note: The "Top" label should be visible after the foam pad is in place. Close the two side sections of the Velcro flaps, followed by the center sections to secure the foam pad.
- The Retroback is now ready for attachment.

## **CLEANING AND DISINFECTING**

**Cleaning and disinfecting are two different processes. Cleaning must precede disinfecting.**

Disassemble the Retroback before cleaning or disinfecting any of its components.

To clean the cover, machine wash in warm water (40° C). Air dry.  

To disinfect the cover, hand wash in warm water using 1/2 cup liquid household bleach per quart of water (125 ml bleach to 1 liter of water) solution and rinse thoroughly, or machine wash in hot water (60° C). Air dry. 

 Follow safety guidelines on bleach container.

To clean the foam pad surface, wipe gently with a cloth dampened with mild soap and water. Air dry. No heat.

To disinfect the foam pad surface, wipe gently with a cloth dampened with a household cleaning disinfectant. Allow foam pad to dry thoroughly before inserting back into cover.

 **Notes:**

**DO NOT** sterilize or autoclave the foam pad or cover.

**DO NOT** immerse the foam pad in water or liquids.

To clean the steel clip, wipe with a cloth dampened with a household cleaner.

To disinfect the steel clip, wipe with a cloth dampened with a household cleaning disinfectant.

For multiple patient use, institutions **MUST** clean and disinfect the product between each patient use.

## **RETURN, REPAIR & CONTACT INFORMATION**

If you require a repair, you may return the product for assessment. Before returning the product to ROHO, Inc., contact our Customer Support Department at 1-800-851-3449 toll free in the **U.S.A.** for a Return Authorization Number. A Return Authorization Number must be obtained from ROHO, Inc. and a completed Return Authorization Form must be included with your product when returned. The Return Authorization Form can be obtained from our website at [www.therohogroup.com](http://www.therohogroup.com) or by contacting Customer Support. **Outside the U.S.A.**, contact the nearest ROHO International distributor. See website for current list of ROHO International distributors ([www.therohogroup.com](http://www.therohogroup.com)).

Be advised that a charge will be assessed for any service provided for a product that does not qualify under the product's warranty.

Failure to submit a Return Authorization Form may result in the product being returned without evaluation or repair.

## **DISPOSAL**

When properly used and disposed, there are no known environmental hazards associated with the components of the Retroback. Dispose of the product and / or components in accordance with the applicable regulations in your jurisdiction.

## Limited Warranty

### What is covered?

ROHO, Inc. covers any defects in materials or workmanship of the product. Any substitutions of supplied components will void the warranty.

### For how long?

12-months from the date the product was originally purchased.

### What we will do.

Within a reasonable amount of time after you return the product to us, we will repair or replace any defect in material and workmanship and ship it back to you, all free of charge.

### What we will not do.

We will not repair a product free of charge if it has been misused, damaged by an accident, or damaged from "an act of Nature", e.g., flood, tornado, earthquake, fire.

### What the customer must do.

Within the **U.S.A.**, the customer must contact ROHO, Inc. to obtain a Return Authorization Number and a completed Return Authorization Form must be included with the product when returned. The Return Authorization Form can be obtained from our website at [www.therohogroup.com](http://www.therohogroup.com) or by contacting Customer Support at 1-800-851-3449 toll free. **Outside the U.S.A.**, contact the nearest ROHO International distributor. See website for current list of ROHO International distributors ([www.therohogroup.com](http://www.therohogroup.com)).

Failure to submit a Return Authorization Form may result in the product being returned without evaluation or repair.

### Questions.

Call our Customer Support Department toll free from **U.S.A.** at (800)-851-3449. **Outside of U.S.A.** call your ROHO International distributor or the ROHO Customer Support Department at 1-618-277-9150.

### Disclaimer

Any implied warranties, including the MERCHANTABILITY and FITNESS FOR A PARTICULAR PURPOSE are also limited to the period of time specified above from the date of original purchase and any action for a breach of such warranties or any express warranties herein must be commenced within this time. ROHO, Inc. shall not be responsible for any incidental or consequential damages with respect to economic loss or injuries to property, whether as a result of a breach of express or implied warranties, or otherwise. ROHO, Inc. will not be responsible for the results of careless handling, unreasonable or improper use of this product.

### Your Rights and the Effect of this Warranty upon Them

This warranty restricts the duration of all implied warranties as noted above and limits or excludes recovery for incidental or consequential damages. Some jurisdictions do not allow limitations on how long implied warranties will last and some jurisdictions do not allow the exclusion or limitation of consequential or incidental damages, so those provisions may not apply to you. While this warranty gives you specific legal rights, you may also have other rights which vary from jurisdiction to jurisdiction.

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MDSS GmbH
of ROHO, Inc. are certified to
Belleville, IL 62221-5429 U.S.A.
U.S.A. & Canada: 800-851-3449, Fax 888-551-3449