
Supplier: These instructions must be given to the user of this product.
Individual or Caregiver: Before using this product, read the instructions and save for future reference.

This document is provided as a supplement to the operating instructions included with your product.

! This ROHO® DRY FLOATATION® custom-made device is designed to meet an individual's needs. Contact the prescribing physician regarding the intended use of this product.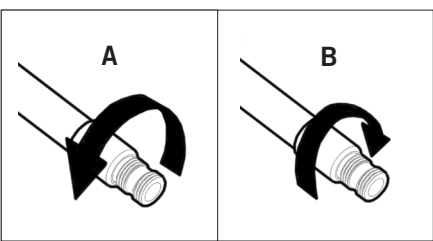

Instructions:

1. Place the product on the surface, air cells facing the individual. NOTE: If provided, straps, snaps, or hook and loop fasteners should be used to secure the product.
2. Open the valve (A). Use the hand pump to inflate the product until it arches slightly.
3. Adjust the inflation level as prescribed. Close the valve (B) when adjustment is complete.
4. Confirm that the individual is in their usual position. Confirm correct placement of the product.

! **CHECK INFLATION!** Confirm that there is **NO CONTACT** between the individual and the base of the product. If there is contact, add air.

! ROHO, Inc. recommends at least ½ in. (1.5 cm) of space between the base of the product and the individual's bony prominences. Follow the prescriber's recommended inflation levels.
ROHO, Inc. has a policy of continual product improvement and reserves the right to amend this document.

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© 2013, 2015 ROHO, Inc.
Products featured in this document may be covered by one or more U.S. and foreign patents and trademarks. For further details please refer to www.roho.com. ROHO® and DRY FLOATATION® are registered trademarks of ROHO, Inc.
ROHO, Inc:n toimintatapaan kuuluu jatkuva tuotekehitys ja se pidättää oikeuden muokata tätä asiakirjaa.
ROHO, Inc. possui uma política de melhoria contínua dos produtos e reserva -se o direito de alterar este documento.