




- ROHO® HIGH PROFILE® Single Compartment Cushion
- ROHO® LOW PROFILE® Single Compartment Cushion
- ROHO® HIGH PROFILE Dual Compartment Cushion
- ROHO® LOW PROFILE Dual Compartment Cushion
- ROHO® MID PROFILE™ Cushion
- ROHO® ENHANCER® Cushion

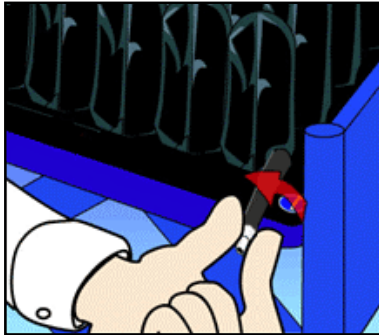

STEP 1: Place cushion on chair, making sure it is centered with air cells up, with air valve in front, left corner (when the user is seated). Consult your prescriber about alternative positions of air valves. Turn valve counterclockwise to open.

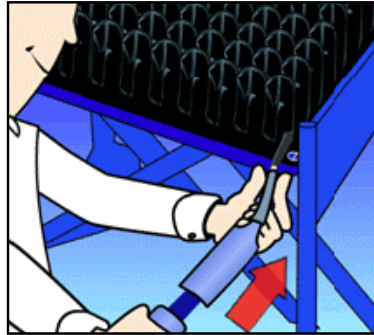


STEP 2: Slide the pump's rubber nozzle over the valve and inflate the cushion until it begins to slightly arch upward.

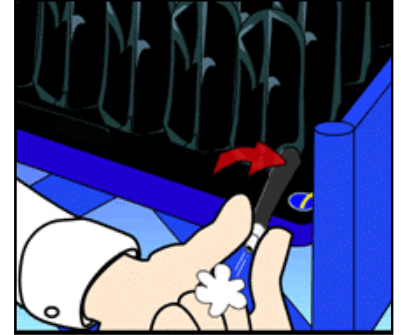

STEP 3: Pinch the pump's nozzle and turn valve clockwise to close. Remove pump. (Repeat steps 1 - 3 for remaining air valves on multi-valve cushions.)



STEP 4: Have the user sit in the chair, making sure the cushion is centered underneath. The user should be seated in their normal sitting position.

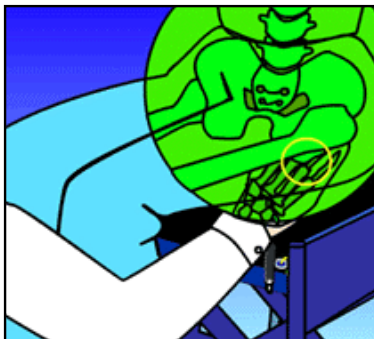

STEP 5: Slide your hand between the cushion's surface and the user's bottom. Lift their leg slightly and feel for their lowest bony prominence. Then lower their leg to a sitting position.



STEP 6: Turn valve counterclockwise to let out air, while keeping your hand under the person's lowest bony prominence. Release air until you can barely move your finger tips – no more than 1 inch (2.5 cm) and no less than 1/2 inch (1.5 cm). Turn valve clockwise to close.

ROHO® Dual Compartment Cushions: To maximize the positioning benefits for individuals with a pelvic obliquity, the side with the deepest bony prominence must be adjusted first. Dual compartments may be used for side-to-side or front-to-back positioning. For those who use front-to-back positioning, adjust the rear section first. Once both adjustments are made, recheck each compartment to ensure proper adjustment.



NOTE: DO NOT sit on an improperly inflated cushion. Under-inflation and over-inflation of the cushion sections reduce or eliminate the cushion's benefits and could increase risk to the skin and other soft tissue. The cushion is most effective when there is 1/2 inch (1.5 cm) to 1 inch (2.5 cm) of air between the user's bottom and the seating surface.


© 2017 Permobil Seating & Positioning. The following are trademarks and registered trademarks of ROHO, Inc: ROHO®, HIGH PROFILE®, MID PROFILE™, LOW PROFILE®, ENHANCER®, QUADTRO SELECT®, ISOFLO®, and CONTOUR SELECT®.

Permobil has a policy of continual product improvement and reserves the right to amend this document. The current version of this document is available on our website at permobilus.com.


ORIGINAL ROHO PRODUCT: Watch for the ROHO logo, yellow rope and unique black ROHO air cells under the cover. If you see these features, you can be sure that you have an original ROHO product.

How to Adjust Your ROHO® Cushion with an ISOFLO® Memory Control


- ROHO® QUADTRO SELECT® HIGH PROFILE® Cushion
- ROHO® QUADTRO SELECT® MID PROFILE™ Cushion
- ROHO® QUADTRO SELECT® LOW PROFILE® Cushion
- ROHO® CONTOUR SELECT® Cushion

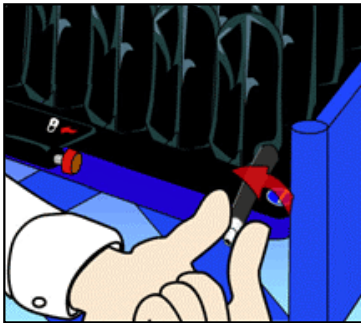

STEP 1: Place SELECT cushion on chair, making sure it is centered and right side up with the ISOFLO Memory Control Unit facing forward (Consult your prescriber about alternative positions of ISOFLO). Turn valve counterclockwise to open.

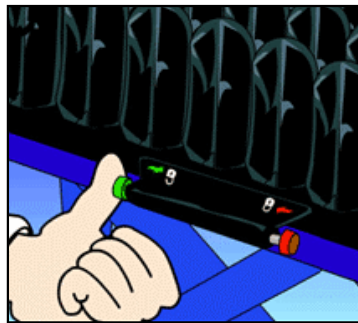

STEP 2 Make sure the ISOFLO Memory Control is also open by pushing ISOFLO's Green Knob to the right or "UNLOCKED".

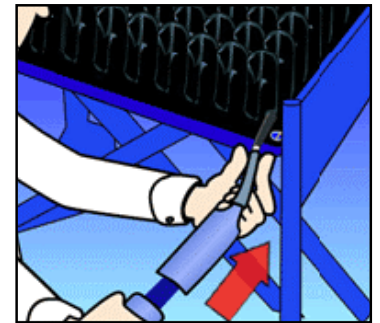


STEP 3 Slide the pump's rubber nozzle over the valve and inflate the SELECT cushion until it begins to slightly arch upward.



STEP 4 Pinch pump's nozzle and turn valve clockwise to close. Remove pump.




STEP 5 Have the user sit in the chair, making sure the cushion is centered underneath. The user should be seated in their normal sitting position.

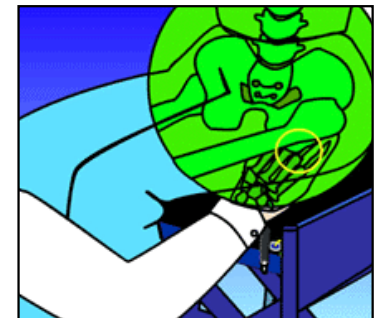

STEP 6 Slide your hand between the cushion's surface and the user's bottom. Lift their leg slightly and feel for their lowest bony prominence. Then lower their leg to a sitting position.



STEP 7 Rotate valve counterclockwise to let out air, while keeping your hand under the person's lowest bony prominence. Release air until you can barely move your finger tips – no more than 1 inch (2.5 cm) and no less than 1/2 inch (1.5 cm). Turn valve clockwise to close.

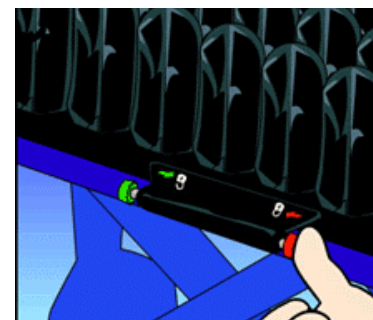

STEP 8 Position user on the cushion in their desired posture and have them maintain this position while air transfers through the ISOFLO Memory Control. Once the air transfer is complete, push the ISOFLO's "Red" knob toward the left, in the LOCKED position. This will isolate the flow of air and provide a more stable sitting position. Recheck each compartment of the cushion to ensure proper inflation.